



Pre-Installation Checklist:

In preparation of receiving your Medrad™ Intego PET Infusion System, your department will need to do the following:

1. Ensure that your Co-57 and Cs-137 sources will be on-site when your Intego arrives. These will be used for calibration during installation.



A few things to remember:

- Sources need to be greater than 100 uCi (3.7 MBq) at the time of calibration with a 3% source uncertainty
- Sources need to be in a 30 cc sealed vial



2. Order disposables and labels for your Medrad™ Intego PET Infusion System.

You will need disposables for your Intego system in order to complete installation and training. You can order your disposables by contacting our Customer Support staff. (phone: 800-633-7231 or email: ricustomercare@bayer.com)

Intego Disposables:	Catalog #:	Quantities:	Product Images:
Source Administration Set (SAS)	INT CSS	Box of 20 Disposables (1 disposable used with each new vial)	
Patient Administration Set (PAS)	INT CPS	Box of 50 Disposables (1 disposable used with each new patient)	
Printer Labels	3017517	700 labels per roll (Proactive refills recommended)	

3. Work with your Bayer HealthCare consultant to coordinate with your radiopharmaceutical provider on the delivery of a 30 mCi vial of F-18 FDG. This will be used to perform linearity during installation.

4. Ensure that a 1,000 cc bag of saline is available for installation and training.



Bayer HealthCare

Radiology
Bayer HealthCare Medical Care

Bayer HealthCare LLC
100 Bayer Boulevard
P.O. Box 915
Whippany, NJ 07981

www.radiologysolutions.bayer.com



Bayer Medical Care Inc.
1 Bayer Drive
Indianola, PA 15051-0780 USA

Customer Service/Orders
1-800-633-7231

Customer Service Fax
1-412-767-4120

Bayer reserves the right to modify the specifications and features described herein, or discontinue manufacture of the product described at any time without prior notice or obligation. Please contact your authorized Bayer HealthCare representative for the most current information.

Bayer, the Bayer Cross, and Medrad™ are trademarks of the Bayer group of companies.

© 2012, 2015 Bayer

PP-INT-US-0013 March 2015